Polyclonal Rabbit Anti-Human IgM/FITC, Rabbit F(ab')₂
Polyclonal Rabbit Anti-Human IgM/RPE, Rabbit F(ab')₂

F0058
R5111

ENGLISH

Intended use

For in vitro diagnostic use.

F0058 and R5111 are intended for use in flow cytometry. In flow cytometry, antibodies to IgM are useful for the demonstration of cell surface IgM, and, thus, the subtyping of pre-B-cell acute lymphoblastic leukaemia (pre-B-ALL), B-cell acute lymphoblastic leukaemia (B-ALL) and B-cell chronic lymphoproliferative disorders together with a panel of other antibodies (1-6). Interpretation of results must be made within the context of the patient's clinical history and other diagnostic tests by a certified professional.

Summary and explanations

Most B cells, with the exception of pre-B progenitor and pre-B cells, and mature plasma cells, express immunoglobulin on their surface. Pre-B cells express cytoplasmic μ-chains but no light chains, whereas the early B lymphocytes express membrane IgM only. The maturing B lymphocytes additionally produce IgD that is inserted into the cell membrane joining IgM and establishing a population of IgM+IgD+ B lymphocytes, which is the largest population of circulating B lymphocytes in man. Subsequent re-arrangement of the constant region of the immunoglobulin heavy chain genes results in cells expressing membrane IgG or IgA, initially in concert with IgM or IgD. Antigen exposure activates the production of these cells. They may evolve from less mature B precursors in a primary response, or from memory cells in a secondary response (7). B cell chronic lymphocytic leukaemia (B-CLL) cells are commonly positive both for surface IgM and IgD (2-6). Moreover, IgM- and IgD-positive cases are frequently seen in B-cell polymphocytic leukaemia, hairy cell leukaemia, splenic lymphoma with villous lymphocytes, mantle cell lymphoma, and follicular cell lymphoma (1, 5). Pre-B-ALL cases express cytoplasmic IgM whereas B-ALL cases express surface IgM (1).

Reagent provided

The Anti-IgM conjugates, F0058 and R5111, have been produced from a F(ab')₂ fragment of affinity-isolated polyclonal rabbit antibody. The conjugates are provided in liquid form in buffer containing 1% bovine serum albumin (BSA) and 15 mmol/L NaN₃, pH 7.2. Each vial of F0058 contains 100 tests (10 μL of F0058 for up to 10⁶ leucocytes from normal human peripheral blood. Each vial of R5111 contains 100 tests (10 μL of conjugate for up to 10⁶ leucocytes from normal human peripheral blood).

Conjugate concentration q/L: See label on vial.

Antibody Code	Fluorochrome	Negative Control Code
F0058	FITC (Fluorescein Isothiocyanate Isomer 1)	X0929
R5111	RPE (R-phycoerythrin)	X0930

Preparation

1. The antibody used for FITC and RPE conjugation has been solid-phase absorbed with human plasma proteins to remove unwanted antibodies.
2. The absorbed antibody has been further purified by affinity chromatography on a column with immobilized human IgM.
3. The affinity-isolated antibody has then been degraded with pepsin and the F(ab')₂ fragment isolated by gel filtration.
4. Finally, the F(ab')₂ fragment has been conjugated with fluorescein isothiocyanate isomer 1 or R-phycoerythrin.

Immunogen

IgM isolated from human serum of a patient with Waldenström's macroglobulinaemia.

Specificity

Anti-IgM reacts with the μ-chains of human IgM. The specificity has been ascertained as described below. To obtain maximum sensitivity, the crossed immunoelectrophoresis and ELISA specificity tests were performed prior to affinity purification and pepsin degradation.

Crossed immunoelectrophoresis: Only the IgM precipitation arch appears when the antibody is run against human plasma. Staining: Coomassie Brilliant Blue.

ELISA: In indirect ELISA no significant reaction is seen with human IgA and IgG. In double antibody sandwich ELISA no significant reaction is observed with human plasma stripped of IgM.

Flow cytometry: When tested on human lysed whole blood, Anti-IgM labels a subpopulation of the CD19-positive cells.

Flow cytometric analysis of peripheral blood lymphocytes demonstrate that Anti-IgM labels a proportion of B-CLL. Thus, in one study of 121 cases (3), 32 were positive for IgM only, while an additional 21 cases were positive for both IgD and IgM. In another study of 165 cases (4), 142 were positive for IgM and a number of these cases were also positive for IgD.

Precautions

1. For professional users.
2. This product contains sodium azide (NaN₃), a chemical highly toxic in pure form. At product concentrations, though not classified as hazardous, sodium azide may react with lead and copper plumbing to form highly explosive build-ups of metal azides. Upon disposal, flush with large volumes of water to prevent metal azide build-up in plumbing.
3. As with any product derived from biological sources, proper handling procedures should be used.
4. Wear appropriate Personal Protective Equipment to avoid contact with eyes and skin.
5. Unused solution should be disposed of according to local, State and Federal regulations.

Storage

Store in the dark at 2-8 °C. Do not use after expiration date stamped on vial. If reagents are stored under any conditions other than those specified, the conditions must be verified by the user. There are no obvious signs to indicate instability of this product. Therefore, positive and negative controls should be run simultaneously with patient specimens. During storage a small precipitate may occasionally develop causing a fine granular non-specific staining. By a simple filtration (0.22 μm cellulose acetate filter), the original high quality of the conjugate will be restored. Conjugates should not be stored in diluted form. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem with the antibody is suspected, contact our Technical Services.

Procedural note

Before staining samples of peripheral blood, the mononuclear cells must be isolated by centrifugation on a separation medium or the blood sample washed to remove the soluble serum proteins. As human monocytes bind serum immunoglobulins via their surface Fc receptors, these cells should be removed by depletion or identified.

Staining procedure

1. Collect venous blood into a test tube containing an anticoagulant.
2. Transfer 100 μL of the anticoagulated blood to the test tube.
3. Add 2 mL of 0.01 mol/L PBS, pH 7.4. Mix gently by using a vortex mixer.
4. Centrifuge at 300 x g for 5 minutes, then aspirate the supernatant, leaving approximately 50 μL of fluid.
5. Repeat steps 3 and 4 two more times.
6. Add 10 μL rabbit immunoglobulin fraction, Code X0903, for blocking. Mix gently by using a vortex mixer and incubate in the dark at 37 °C for 30 minutes.
7. Add 10 μL of fluorochrome-conjugated Anti-IgM. Mix gently.
8. Use a non-reactive F(ab')₂ fragment of rabbit immunoglobulins, and conjugated with the same fluorochrome, as a negative control (see table).

(102448-002)

F0058/R5111/EFG/ROP/01.02.07 p. 1/4

9. Incubate in the dark at 4 °C for 30 minutes.
10. Add 1-2 mL of erythrocyte-lysing reagent to each tube and mix gently. Follow the reagent manufacturer's recommendations for time and temperature of incubation.
11. Centrifuge at 300 x g for 5 minutes.
12. Aspirate the supernatant, leaving approximately 50 μL of fluid.
13. Add 2 mL 0.01 mol/L PBS, pH 7.4. Mix gently by using a vortex mixer.
14. Repeat steps 11 and 12.
15. Resuspend pellet in an appropriate fluid for flow cytometry, e.g. 0.3 mL 1% paraformaldehyde (fixative) in 0.01 mol/L PBS, pH 7.4.
16. Analyse on a flow cytometer.

It is recommended to include a suitable positive and negative control sample with each run for reagent and preparation control. Note that fluorochrome conjugates are light sensitive, and samples should be protected from light during the staining procedure and until the analysis. For staining of cytoplasmic IgM Dako IntraStain, Fixation and Permeabilisation Kit, Code K2311 is recommended.

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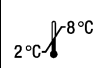
- van Dongen JJM, Adriaansen HJ. Immunobiology of leukaemia. In: Henderson ES, Lister TA, Greaves MF, editors. Leukemia. Philadelphia, London, Toronto, Montreal, Sydney, Tokyo: WB Saunders Company; 1996. p. 83-130.
- Geisler CH, Larsen JK, Hansen NE, Hansen MM, Christensen BE, Lund B, et al. Prognostic importance of flow cytometric immunophenotyping of 540 consecutive patients with B-cell chronic lymphocytic leukemia. Blood 1991;78:1795-1802.
- Cartron G, Linassier C, Bremond JL, Desablens B, Georget MT, Fimbel B, et al. CD5 negative B-cell chronic lymphocytic leukemia: clinical and biological features of 42 cases. Leuk Lymphoma 1998;31:209-16.
- Gandini D, Lanza F, Latorraca A, Levato F, Del Senno L, Castoldi G. Immunophenotypic and genotypic characterization of B-cell chronic lymphocytic leukemia patients from Northern Italy. Haematologica 1993;78:18-24.
- Batata A, Shen B. Immunophenotyping of subtypes of B-chronic (mature) lymphoid leukemia. A study of 242 cases. Cancer 1992;70:2436-43.
- Shen PUF, Fuller SG, Rezuke WN, Sherburne BJ, DiGiuseppe JA. Laboratory, morphologic, and immunophenotypic correlates of surface immunoglobulin heavy chain isotype expression in B-cell chronic lymphocytic leukemia. Am J Clin Pathol 2001;116:905-12.
- Deegan MJ. B lymphocytes and plasma cells: their development and identification. In: Keren DF, editor. Flow cytometry in clinical diagnosis. Chicago: ASCP Press; 1989. p. 139-63.

REF	<p>Catalogue number Référence du catalogue Bestellnummer</p>	 <p>Temperature limitation Limites de température Zulässiger Temperaturbereich</p>	 <p>Use by Utiliser jusque Verwendbar bis</p>
IVD	<p>In vitro diagnostic medical device Dispositif médical de diagnostic in vitro In-Vitro-Diagnostikum</p>	 <p>Keep away from sunlight (consult storage section) Conserver à l'écart du soleil (se reporter à la section conservation) Lichtgeschützt lagern (siehe Abschnitt zur Lagerung)</p>	 <p>Manufacturer Fabricant Hersteller</p>
LOT	<p>Consult instructions for use Consulter les instructions d'utilisation Gebrauchsanweisung beachten</p>	<p>Batch code Code du Lot Chargenbezeichnung</p>	

(102448-002)

F0058/R5111/EFG/ROP/01.02.07 p. 4/4