MultiMix™
Dual-Colour Reagent
Anti-Human CD5/FITC
Anti-Human CD19/RPE
Code No./ Code/ Code-Nr. FR882

ENGLISH	
Intended use	For in vitro diagnostic use. FR882 is intended for use in flow cytometry. Antibodies to CD5 and CD19 are considered essential for the initial evaluation of acute and chronic lymphoproliferative disorders as well as acute leukaemias (1). Interpretation of results must be made within the context of the patient's clinical history and other diagnostic tests by a certified professional.
Reagent provided	FR882 comprises the following two, carefully matched, fluorescent antibodies: Monoclonal Mouse Anti-Human CD5, Clone DK23, conjugated with fluorescein isothiocyanate isomer 1 (FITC). Monoclonal Mouse Anti-Human CD19, Clone HD37, conjugated with R-phycoerythrin (RPE). The two conjugates in FR882 have been produced from purified monoclonal mouse antibodies of isotype IgG1, kappa. FR882 is provided in liquid form in buffer containing 1% bovine serum albumin (BSA) and 15 mmol/L NaN ₃ , pH 7.2. Each vial contains 50 tests (10 µL of conjugate for up to 10 ⁶ leucocytes from normal human peripheral blood).
Specificity	The specificity of Anti-CD5, DK23, is equivalent to that of the CD5-clustered antibody Leu-1 (2). In chronic leukaemia and myeloma, Anti-CD5, DK23, has been shown to label 4/4 (100%) T-cell chronic lymphocytic leukaemias, 2/2 (100%) T-cell prolymphocytic leukaemias, 36/39 (92%) B-cell chronic lymphocytic leukaemias, 5/6 (83%) B-cell prolymphocytic leukaemias, 0/14 (0%) hairy cell leukaemias, and 0/25 (0%) myelomas (3). Anti-CD19, HD37, was included in the Second, Third, Fourth and Fifth International Workshops and Conferences on Human Leucocyte Differentiation Antigens and studies by a number of laboratories confirmed its reactivity with CD19 (4, 5). Anti-CD19, HD37, labels human B cells in peripheral blood, bone marrow and other tissues. Additionally, B-cell lymphoproliferative disorders gave positive reactions with the antibody, i.e. acute lymphoblastic leukaemia, chronic lymphocytic leukaemia, hairy cell leukaemia, lymphoblastic lymphoma (Burkitt type), centroblastic/centrocytic lymphoma, and diffuse non-Hodgkin's lymphoma. The antibody was shown to be unreactive with other cells in the human haematopoietic system and did not react with non-haematopoietic cells, e.g. in kidney, liver, breast or lung tissues (5).
Precautions	1. For professional users. 2. This product contains sodium azide (NaN ₃), a chemical highly toxic in pure form. At product concentrations, though not classified as hazardous, sodium azide may react with lead and copper plumbing to form highly explosive build-ups of metal azides. Upon disposal, flush with large volumes of water to prevent metal azide build-up in plumbing. 3. As with any product derived from biological sources, proper handling procedures should be used.
Storage	Store in the dark at 2-8 °C. Do not use after expiration date stamped on vial. If reagents are stored under any conditions other than those specified, the conditions must be verified by the user. There are no obvious signs to indicate instability of this product. Therefore, positive and negative controls should be run simultaneously with patient specimens. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem with the antibody is suspected, contact our Technical Services.
Staining procedure	1. Transfer 100 µL of anticoagulated (EDTA) blood to a 12 mm x 75 mm polystyrene test tube. 2. Add 10 µL of FR882 and mix gently by using a vortex mixer. 3. Incubate the tube in the dark at 2-8 °C for 30 minutes or at room temperature (20-25 °C) for 15-30 minutes. 4. Add 100 µL of Uti-Lyse™ Reagent A (DakoCytomation code No. S3325) to the tube and mix gently by using a vortex mixer. Incubate the tube for 10 minutes at room temperature in the dark. 5. Add 1 mL of Uti-Lyse™ Reagent B (DakoCytomation code No. S3325) to the tube and mix gently by using a vortex mixer. Incubate the tube for 10 minutes at room temperature in the dark. 6. Centrifuge the tube at 300 x g for 5 minutes. Gently aspirate the supernatant and discard it leaving approximately 50 µL of fluid in the tube. 7. Add 2 mL of PBS (DakoCytomation code No. S3024) to the tube and resuspend the cells by using a vortex mixer. 8. Repeat step 6. 9. Resuspend the cells in an appropriate fluid for flow cytometry, e.g. 0.3 mL PBS. 10. Analyse the sample on a flow cytometer or store it in the dark at 2-8 °C until analysis. Samples should be analysed within 24 hours after staining. Note that fluorochrome conjugates are light sensitive, and samples should be protected from light during the staining procedure and until the analysis.
Procedural notes	Step 1: It is optional to include a suitable positive and negative control sample with each run for reagent and preparation control. Step 2: The volume of conjugate recommended is a guideline only. Optimal staining conditions may vary depending on specimen and preparation method and should be determined by each individual laboratory. It is optional to include a control reagent test tube. The control reagent should match the isotypes and fluorochromes of the conjugated antibody reagent. The recommended Dual-Colour Control Reagent for FR882 is DakoCytomation code No. X0932. Steps 4 and 5: If another cell-lysing reagent is used, please follow the recommendations for that reagent. Note that if the alternative lysing reagent does not contain fixative, such as DakoCytomation EasyLyse™, code No. S2364, the PBS in step 9 should contain 1% paraformaldehyde unless the sample is analysed within the time frame recommended for the lysing reagent. Step 10: Abnormal numbers of cells expressing the target antigen or cells showing aberrant expression levels of the antigen can be expected in some disease states. This may result in an altered pattern of staining. It is important to understand the normal expression pattern of an antigen and its relationship to the expression of other relevant antigens in order to perform appropriate analysis. Multicolour reagents are preferable to single-colour reagents for the comprehensive analysis of flow cytometry specimens. The correct use of colour compensation is particularly important in multicolour analysis.

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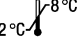
- Braylan RC, Orfao A, Borowitz MJ, Davis BH. Optimal number of reagents required to evaluate hematolymphoid neoplasias: results of an international consensus meeting. Cytometry 2001;46:23-7.
- Stein H, Lennert K, Feller AC, Mason DY. Immunohistological analysis of human lymphoma: correlation of histological and immunological categories. Adv Cancer Res 1984;42:67-147.
- Erber WN, Mynheer LC, Mason DY. APAAP labelling of blood and bone-marrow samples for phenotyping leukaemia. Lancet 1986;i:761-5.
- Nadler LM. B cell/leukemia panel workshop: summary and comments. In: Reinherz EL, Haynes BF, Nadler LM, Bernstein ID, editors. Leukocyte typing II. Proceedings of the 2nd International Workshop on Human Leukocyte Differentiation Antigens; 1984 Sept 17-20; Boston, USA. New York, Berlin, Heidelberg, Tokyo: Springer-Verlag; 1986. Volume 2. p. 3-43.
- Mason DY, Ladyman H, Gatter KC. Immunohistochemical analysis of monoclonal anti-B cell antibodies. In: Reinherz EL, Haynes BF, Nadler LM, Bernstein ID, editors. Leukocyte typing II. Proceedings of the 2nd International Workshop on Human Leukocyte Differentiation Antigens; 1984 Sept 17-20; Boston, USA. New York, Berlin, Heidelberg, Tokyo: Springer-Verlag; 1986. Volume 2. p. 245-55.

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