Isotype Reagent
Mouse IgG2a/FITC
Mouse IgG2a/RPE

Code X0933
Code X0950

ENGLISH

Intended use	For in vitro diagnostic use. X0933 and X0950 are intended for use in flow cytometry. These reagents have been produced especially as negative controls for Dako FITC- or RPE-conjugated monoclonal mouse antibodies of isotype IgG2a.
Introduction	In flow cytometry, isotype-specific negative control reagents typically are fluorochrome-conjugated monoclonal antibodies directed against non-human antigens. These reagents measure the amount of conjugate binding that is unrelated to the target antigen of the human cell. This non-immunological binding may be caused by Fc receptors on the cells or by the physico-chemical characteristics of the fluorochrome conjugate. A conjugated monoclonal mouse antibody selected as an isotypic control should be of the same isotype as the conjugated monoclonal mouse antibody used as the test reagent.
Reagent provided	X0933 and X0950 are purified monoclonal mouse antibody, clone DAK-GO5, conjugated with fluorescein isothiocyanate 1 (FITC) or R-phycoerythrin (RPE). The conjugates are provided in liquid form in buffer containing 1% bovine serum albumin (BSA) and 15 mmol/L NaN ₃ , pH 7.2. Each vial contains 100 tests (10 µL of conjugate for up to 10 ⁶ cells from normal human peripheral blood). <u>Isotype:</u> IgG2a, kappa. <u>Conjugate concentration mg/L:</u> See label on vial.
Specificity	The antibody produced by clone DAK-GO5 is directed towards <i>Aspergillus niger</i> glucose oxidase, an enzyme which is neither present nor inducible in mammalian tissues.
Precautions	1. For professional users. 2. This product contains sodium azide (NaN ₃), a chemical highly toxic in pure form. At product concentrations, though not classified as hazardous, sodium azide may react with lead and copper plumbing to form highly explosive build-ups of metal azides. Upon disposal, flush with large volumes of water to prevent metal azide build-up in plumbing. 3. As with any product derived from biological sources, proper handling procedures should be used. 4. Wear appropriate Personal Protective Equipment to avoid contact with eyes and skin. 5. Unused solution should be disposed of according to local, State and Federal regulations.
Storage	Store in the dark at 2-8 °C. Do not use after expiration date stamped on vial. If reagents are stored under any conditions other than those specified, the conditions must be verified by the user. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem with the reagent is suspected, contact Dako Technical Services.
Dilution guideline	In flow cytometry, X0933 and X0950 may be used at a volume of 10 µL per test when adjusted to the same conjugate concentration as the test antibody. This is guideline only. The optimal amount of the negative control may vary, e.g. depending on the specimen and preparation method, and should be determined by each individual laboratory. Note that fluorochrome conjugates are light sensitive, and samples should be protected from light during the staining procedure and until the analysis.

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Agilent Technologies Singapore (International) Pte Ltd.