**Negative Control  
Mouse IgM/FITC  
Code No./ Code/ Code-Nr. X 0934**  
Edition/ Ausgabe 01.11.02

ENGLISH	
<b>Intended use</b>	For in vitro diagnostic use. X 0934 is intended for use in flow cytometry. X 0934 has been produced especially as a negative control for DakoCytomation FITC-conjugated monoclonal mouse antibodies of isotype IgM.
<b>Introduction</b>	In flow cytometry, isotype-specific negative control reagents typically are fluorochrome-conjugated monoclonal antibodies directed against non-human antigens. These reagents measure the amount of conjugate binding that is unrelated to the target antigen of the human cell. This non-immunological binding may be caused by Fc receptors on the cells or by the physico-chemical characteristics of the fluorochrome conjugate. A conjugated monoclonal mouse antibody selected as an isotypic control should be of the same isotype as the conjugated monoclonal mouse antibody used as the test reagent.
<b>Reagent provided</b>	X 0934 is a purified monoclonal mouse antibody, clone DAK-GO8, conjugated with fluorescein isothiocyanate isomer 1 (FITC). The conjugate is provided in liquid form in buffer containing 1% bovine serum albumin (BSA) and 15 mmol/L NaN <sub>3</sub> , pH 7.2. Each vial contains 100 tests (10 µL of conjugate for up to 10 <sup>6</sup> leucocytes from normal human peripheral blood). <u>Isotype:</u> IgM, kappa. <u>Conjugate concentration mg/L:</u> See label on vial.
<b>Specificity</b>	The antibody produced by clone DAK-GO8 is directed towards <i>Aspergillus niger</i> glucose oxidase, an enzyme which is neither present nor inducible in mammalian tissues.
<b>Precautions</b>	1. For professional users. 2. This product contains sodium azide (NaN <sub>3</sub> ), a chemical highly toxic in pure form. At product concentrations, though not classified as hazardous, sodium azide may react with lead and copper plumbing to form highly explosive build-ups of metal azides. Upon disposal, flush with large volumes of water to prevent metal azide build-up in plumbing. 3. As with any product derived from biological sources, proper handling procedures should be used.
<b>Storage</b>	Store in the dark at 2-8 °C. Do not use after expiration date stamped on vial. If reagents are stored under any conditions other than those specified, the conditions must be verified by the user. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem with the antibody is suspected, contact our Technical Services.
<b>Dilution guideline</b>	In flow cytometry, X 0934 may be used at a volume of 10 µL when adjusted to the same conjugate concentration as the test antibody. However, the optimal amount of X 0934 may vary depending on the specimen and preparation method, and should be determined by each individual laboratory. Note that fluorochrome conjugates are light sensitive, and samples should be protected from light during the staining procedure and until the analysis.

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