MultiMix™
Triple-Colour Reagent
Anti-Human MPO/FITC
Anti-Human CD79 α cy/RPE
Anti-Human CD3/APC
Code No./ Code/ Code-Nr. TC667
ENGLISH

Intended use	<p>For in vitro diagnostic use.</p> <p>TC667 is intended for use in flow cytometry. The reagent is intended for use in identification of myeloperoxidase (MPO) in granulocytes and in monocytes, CD79αcy-positive B cells and, CD3-positive T cells. MPO is a lysosomal enzyme present in neutrophil granulocytes and at low levels in monocytes. Expression of CD79 is largely restricted to B-lineage cells. CD3 is a pan-T-cell antigen and a marker for normal and neoplastic T cells. TC667 is useful for the identification of intracellular MPO, CD79α and CD3. Investigation of surface expression of CD79α and CD3 should be performed separately. Interpretation of results must be made within the context of the patient's clinical history and other diagnostic tests by a certified professional.</p>
Reagent provided	<p>TC667 comprises the following three matched fluorescent antibodies:</p> <p>Monoclonal Mouse Anti-Human Myeloperoxidase, Clone MPO-7, conjugated with fluorescein isothiocyanate isomer 1 (FITC).</p> <p>Monoclonal Mouse Anti-Human CD79αcy, Clone HM57, conjugated with R-phycoerythrin (RPE).</p> <p>Monoclonal Mouse Anti-Human CD3, Clone UCHT1, conjugated with allophycocyanin (APC).</p> <p>The three conjugates in TC667 have been produced from purified monoclonal mouse antibodies of isotype IgG1, kappa. TC667 is provided in liquid form in buffer containing 1% bovine serum albumin (BSA) and 15 mmol/L NaN₃, pH 7.2. Each vial contains 50 tests (20 μL of conjugate for up to 10⁶ leucocytes from normal human peripheral blood).</p>
Specificity	<p>In flow cytometry Anti-Myeloperoxidase, MPO-7, strongly labels the cytoplasm of neutrophil granulocytes in peripheral blood. Monocytes are weakly positive while lymphocytes are unreactive. In bone marrow, cells of the granulocyte lineage are labelled. Anti- Myeloperoxidase, MPO-7, also labels the human promyelocyte cell line HL-60, while other myeloid and lymphoid cell lines are consistently unreactive (1).</p> <p>Anti-CD79αcy, HM57, was clustered as anti-CD79α at the Fifth International Workshop on Human Leucocyte Differentiation Antigens (2).</p> <p>Anti-CD3, UCHT1, was included in the First and Third International Workshops and Conferences on Human Leucocyte Differentiation Antigens, and studies by a number of laboratories confirmed its reactivity with CD3 (3). Anti-CD3, UCHT1, reacts with the ϵ-chain of CD3 (4).</p>
Precautions	<ol style="list-style-type: none"> For professional users. This product contains sodium azide (NaN₃), a chemical highly toxic in pure form. At product concentrations, though not classified as hazardous, sodium azide may react with lead and copper plumbing to form highly explosive build-ups of metal azides. Upon disposal, flush with large volumes of water to prevent metal azide build-up in plumbing. As with any product derived from biological sources, proper handling procedures should be used.
Storage	<p>Store in the dark at 2-8 °C. Do not use after expiration date stamped on vial. If reagents are stored under any conditions other than those specified, the conditions must be verified by the user. There are no obvious signs to indicate instability of this product. Therefore, positive and negative controls should be run simultaneously with patient specimens. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem with the antibody is suspected, contact our Technical Services.</p>
Staining procedure	<ol style="list-style-type: none"> Transfer 50 μL (up to 10⁶ cells) of anticoagulated (EDTA) whole blood, bone marrow or mononuclear cells to a 12 mm x 75 mm polystyrene test tube. Add 100 μL Dako IntraStain, Reagent A (Fixation), code No. K2311. Mix gently with a vortex mixer to ensure that the cells are in suspension. Incubate at room temperature for 15 minutes. Add 2 mL PBS (code No. S3024) and mix gently with a vortex mixer. Centrifuge at 300 x g for 5 minutes, then aspirate the supernatant, leaving approximately 50 μL of fluid. Mix thoroughly with a vortex mixer to ensure that the cells are in suspension and add 100 μL Dako IntraStain, Reagent B (Permeabilization), code No. K2311. Add 20 μL TC667 and mix gently with a vortex mixer to ensure that the cells are in suspension. The 20 μL is a guideline only; the optimal volume should be determined by the individual laboratory. The recommended negative control for TC667 is Dako code No. X0978, a FITC, RPE and APC-conjugated reagent of the same isotype as TC667. Add 20 μL of X0978 to the sample and mix gently with a vortex mixer. The 20 μL is a guideline only; the optimal volume should be determined by the individual laboratory. Incubate in the dark at room temperature for 15 minutes. Repeat steps 4 and 5. Resuspend pellet in an appropriate fluid for flow cytometry, e.g. 0.3 mL PBS (code No. S3024). Analyse on a flow cytometer. If the samples are not analysed immediately, they may be stored in the dark at 2-8 °C for up to 8 hours. <p>Optimal conditions may vary depending on specimen and preparation method, and should be determined by each individual laboratory. It is recommended to include a suitable positive and negative control sample with each run for reagent and preparation control. Note that fluorochrome conjugates are light sensitive, and samples should be protected from light during the staining procedure and until the analysis.</p>

Abnormal numbers of cells expressing the target antigens or aberrant expression levels of the antigens can be expected in some disease states. This may result in an altered pattern of staining. It is important to understand the normal expression pattern for the antigens and their relationship to expression of other relevant antigens in order to perform appropriate analysis.

Multicolour reagents are preferable to single-colour analysis for the comprehensive analysis of neoplastic cells of leukaemia and lymphoma. The correct use of colour compensation is particularly important in multicolour analysis. The combination of FITC, RPE and APC is especially easy to work with since there is minimal spectral overlap in the emission spectra of these fluorochromes.

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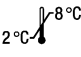
1. Nakase K, Sartor M, Bradstock K. Detection of myeloperoxidase by flow cytometry in acute leukemia. *Cytometry* 1998;34:198-202.
2. Engel P, Wagner N, Tedder TF. B23. CD79 workshop report. In: Schlossman SF, Bousmell L, Gilks W, Harlan JM, Kishimoto T, Morimoto C, et al., editors. *Leucocyte typing V. White cell differentiation antigens. Proceedings of the 5th International Workshop and Conference; 1993 Nov 3-7; Boston, USA.* Oxford, New York, Tokyo: Oxford University Press; 1995. Volume 1. p. 667-70.
3. McMichael AJ, Gotch FM. T-cell antigens: new and previously defined clusters. In: McMichael AJ, Beverley PCL, Cobbold S, Crumpton MJ, Gilks W, Gotch FM, et al., editors. *Leucocyte typing III. White cell differentiation antigens. Proceedings of the 3rd International Workshop and Conference; 1986 Sept 21-26; Oxford, England.* Oxford, New York, Tokyo: Oxford University Press; 1987. p. 31-62.
4. Tunnacliffe A, Olsson C, Traunecker A, Krissansen GW, Karjalainen K, de la Hera A. T3.2. The majority of CD3 epitopes are conferred by the epsilon chain. In: Knapp W, Dörken B, Gilks WR, Rieber EP, Schmidt RE, Stein H, et al., editors. *Leucocyte typing IV. White cell differentiation antigens. Proceedings of the 4th International Workshop and Conference; 1989 Feb 21-25; Vienna, Austria.* Oxford, New York, Tokyo: Oxford University Press; 1989. p. 295-6.

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Agilent Technologies Singapore (International) Pte Ltd.